



## Attention

Do not allow the conductive garments to touch together when turning on or making adjustments to the intensity of the ReBuilder® unit.

It is important to carefully un-snap the lead wire from the conductive garment after use. Dry the snap connectors and the snaps on the garments before putting them away. Doing this will help maintain optimal function of the conductive garments.



Even if you have used Conductive Gloves before, please read these instructions completely as information may have changed.

## Questions

For questions about this product



Contact us toll-free (866) 725-2202



Visit us on the Web  
[www.ReBuilderMedical.com](http://www.ReBuilderMedical.com)



Or email us  
[help@ReBuilderMedical.com](mailto:help@ReBuilderMedical.com)

## Warranty Information

Conductive Gloves carry a 90 day warranty against manufacturer defects. All warranty claims are to be processed by the manufacturer directly and must be accompanied by a receipt for purchase or copy of a dated order to dispense by a certified physician's office.

DO NOT RETURN THIS ITEM TO YOUR DOCTOR FOR WARRANTY RELATED ISSUES.

Contact ReBuilder Medical for all warranty claims  
(877) 717-5487.



636 Treeline Drive  
Charles Town, WV 25414  
Toll Free: (877) 717-5487  
Phone: (304) 725-2202  
Fax: (304) 699-0055  
V030617



## Universal Conductive Garment Gloves

For use with Electrical Stimulators



General Description  
Instructions  
and Warranty Information

Please read all information in this booklet before beginning treatment with the Conductive Gloves.

Verify contents of your package:

- A. Conductive Garment Gloves with attached snap connectors
- B. Electrolyte



## Using the Conductive Gloves

1. Turn the Conductive Gloves inside out.
2. Holding the Electrolyte bottle about 6 inches away, spray each Conductive Glove.
3. Apply the Electrolyte evenly across the entire Conductive Glove until fabric is moist.
4. Turn the Conductive Gloves right-side out.
5. Repeat spray to the Conductive Gloves as necessary to maintain desired moisture level for your treatments.

## Putting on the Conductive Gloves

Slide the Conductive Gloves onto each hand. To reduce stretching, try to avoid “pulling” the gloves on.



## Connecting the snaps

Snap connectors will be connected to the Conductive Gloves. Attach the lead wire from the stimulator into the snap connector. (See fig. 1) You can remove the snaps if it makes it easier to attach the lead wire. (See fig. 2)



It does not matter if the snap is on the inside or the outside of your wrist.

Color of snap may vary.

## How to position your hands using the Conductive Gloves

Sit with your arms on a table or resting across your lap. Do not let the conductive gloves touch each other. If the gloves touch, then the stimulator signal will not travel through your nerves.

## Connecting to the stimulator

Attach your existing lead wire into the top of your stimulator.

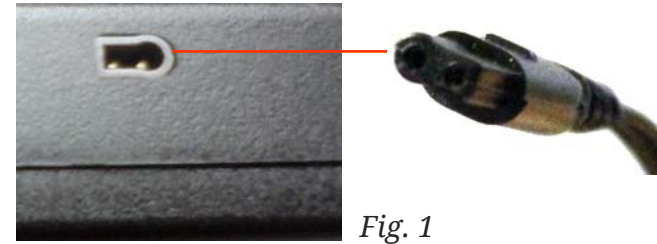


Fig. 1

Insert the metal prongs of the lead wire into each of the snap connectors.



Fig. 2

## Care and Cleaning

Your Conductive Gloves are intended to be used wet. It is suggested you gently hand wash the Conductive Gloves with mild soap once a week or as necessary. Do not machine wash or dry.

These products are supplied as extra products for your convenience. Many physicians will recommend these garments with use of electrical stimulation devices. These garments may assist in the treatment for pain but are not necessary for use with ReBuilder products.