



Attention

Please follow your physician's instructions on how to use this and any other product.



The color of the Universal Twin-Compartment Bucket may be different than in images shown. (Blue Footbath is show in photos for better resolution)

Questions

For questions about this product



Contact us toll-free (866) 725-2202



Visit us on the Web
www.ReBuilderMedical.com



Or email us
help@ReBuilderMedical.com

Warranty Information

All warranty claims are to be processed by the manufacturer directly and must be accompanied by a receipt for purchase or copy of a dated order to dispense by a certified physician's office.

DO NOT RETURN THIS ITEM TO YOUR DOCTOR FOR WARRANTY RELATED ISSUES.
Contact ReBuilder Medical for all warranty claims
(877) 717-5487.



636 Treeline Drive
Charles Town, WV 25414
Toll Free: (877) 717-5487
Phone: (304) 725-2202
Fax: (304) 699-0055
V030617



Universal Twin-Compartment Bucket



General Description
Instructions
and Warranty Information

To use the Twin-Compartment Footbath, you will need the following items:

- A. Universal Twin-Compartment Bucket
- B. Electrolyte
- C. Universal Black non-adhesive pads with attached lead wire



OPTIONAL: You may find that the flip-top squirter lid is more practical for your application. The sprayer top that comes on the bottle can easily be replaced by unscrewing it and screwing the squirter lid on in its place if desired.



Using the Universal Twin-Compartment Bucket


Add 1 teaspoon of appropriate electrolyte to each side of the twin-compartment bucket.



Fill the bucket with enough warm water to reach your ankles when you place your feet into each side of the bucket.



Using the non-adhesive pads with the attached lead wire, place one pad into each side of the bucket. (The pads do not have to be touching your feet.)

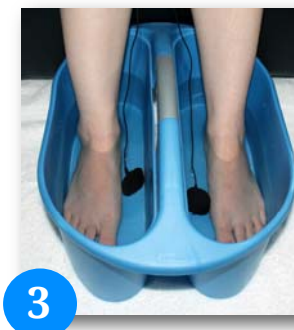

The pads will likely move around during the treatment. The pads transfer the signal to the water and the water transfers the signal to your feet.



Whichever method of application you use, (bucket, conductive socks, or self-adhesive pads) if this is your first time using your stimulator, follow the treatment guide found in the operating and treatment instruction guide for your unit. If you have been using the your unit for a while and are switching to this method from conductive socks or the adhesive pads, you can continue to use your unit with the same time and intensity setting you would normally use.

Attach the lead wire to your stimulator. Turn the stimulator on to the appropriate setting (see note above).



Then just sit back and relax while the device goes to work.

When finished your treatment session, thoroughly dry your feet. Rinse and clean the bucket and the non-adhesive pads to remove any skin debris, oils, and to prevent build-up of minerals.