

All warranty claims are to be processed by the manufacturer directly and must be accompanied by a receipt for purchase or copy of a dated order to dispense by a certified physician's office.

DO NOT RETURN THIS ITEM TO YOUR DOCTOR FOR WARRANTY RELATED ISSUES.

For questions concerning warranty or mechanical or operational problems with your ReBuilder® unit or accessories, please call toll free 877-717-5487.

Neuropathy pain and symptoms can be vastly different for each person, and not all ReBuilder® users will have the same experience during use or experience the same results after use.

Your use of the ReBuilder® will be based on your comfort and experience and may not exactly match the daily outline in this booklet. It is normal to need to take a day or two break from treatment occasionally. It is normal to experience changes in symptoms or experience new or different sensations from time to time. It is normal for symptoms to fluctuate during the weeks of treatment. There are tens of thousands of happy ReBuilder® users, all of them unique, but with one thing in common: Relief.

If you need assistance with use, have technical questions or just need more information, you may contact us on our website [www.ReBuilderMedical.com](http://www.ReBuilderMedical.com) or by calling toll free 877-717-5487.



## THE REBUILDER® SYSTEM







### ReBuilder® Model 2407

ReBuilder Medical Inc.

636 Treeline Drive, Suite A Charles Town, WV  
25414 Phone: 304-725-2202 Fax: 304-725-4915

# GET TO KNOW YOUR REBUILDER®

Confirm all the following items are included with your ReBuilder® kit.

- A**  A. ReBuilder® 2407 Electronic Unit
- B**  B. Set of lead wires
- C**  C. 2 sets of adhesive stimulating pads
- D**  D. A/C Adapter




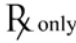





Your ReBuilder® can operate two sets of lead wires and two sets of pads simultaneously. You can use the 1 set of adhesive stimulating pads on your feet while simultaneously using another set of adhesive stimulating pads on your hands, back or shoulders. Do not use both output one and output two on your feet at the same time.

## NOTE!

The power and intensity switch #1 only controls output #1. The power and intensity switch #2 only controls output #2.

This device is part of a complete treatment system that is designed to function in harmony. Do not use with any accessories or attachments that are not recommended by the manufacturer. Do not use in any way other than recommended by the manufacturer. Doing so may result in altered electrical emissions from the device (malfunction) and patient injury or damage to the ReBuilder may occur.

## Summary Glossary

-  Serial Number
-  Non-sterile
-  Not for general waste
-  Caution: Federal (USA) law restricts this device to sale by or on the order of a Licensed Healthcare Provider
-  Keep dry
-  Consult instructions
-  Warning: electricity
-  Manufacturer
-  Contraindications: Use of the ReBuilder is contraindicated for: individuals with pacemakers or any other implanted medical devices, for pregnant women, and for children under 18 years of age. Use only under direct supervision of your physician or health care provider if any of the above conditions apply. Do not use directly over an open wound. Follow your physician's advice for specific instructions.